

C141
ALLPORT

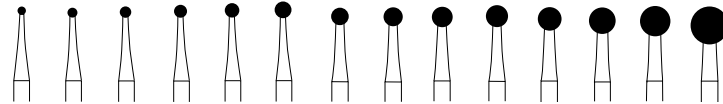 max. 3.000
 Shank 316, 160.000
 3

Shank Size	L mm	ISO	Order No.	010	012	014	016	018	021	023	025	027	029	031	035	040	050
HPL	500	105	001 291...									027	029	031	035	040	050
HP	500	104	001 291...	010		014		018		023	025	027	029	031	035	040	050
RAXL	500	206	001 291...	010		014		018		023	025	027	029	031	035	040	050
RAL	500	205	001 291...	010		014		018		023	025	027	029	031	035	040	050
FGXL	500	316	001 291...	010	012	014	016	018	021	023							

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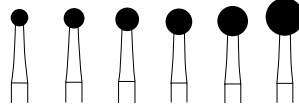The optimum drill for funnel-shaped opening of all pulp chambers.

Advantages: Special drill with rounded tip (GK) for opening and extending the pulp chamber.

The rounded, blunt tip (GK) prevents inadvertent perforation of the pulp chamber floor and root walls.

Vibration-free preparation produced by the spiral-shaped cutting edges, which ensure rapid removal of the dentine.

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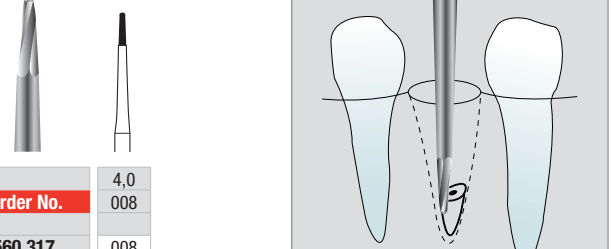
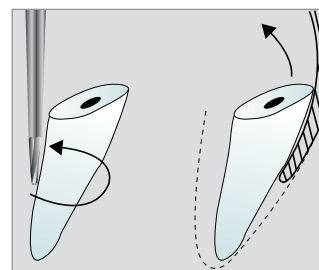
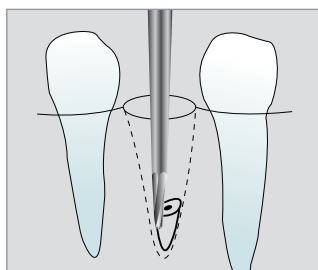
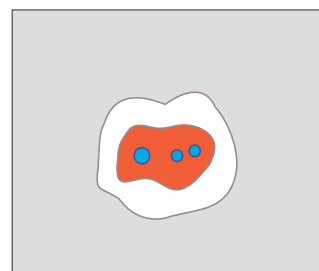
Efficiently cutting special endodontic bur used for detecting all root canals in the case of multirouted teeth. The H1SNL has a particularly high cutting efficiency and its long neck ensures a clear view in deeper regions of the cavity for opening the access cavity of the root canal system. Use in a contra-angle with micromotor at 1,500 – 2,000 rpm, without cooling, dry and apply only light pressure during preparation.

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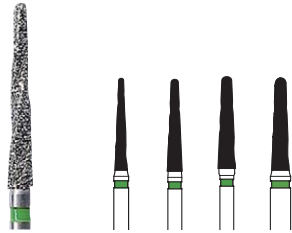
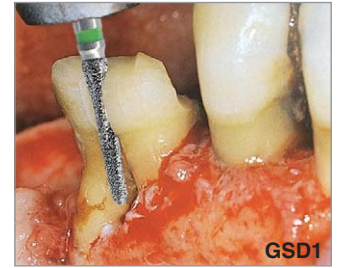
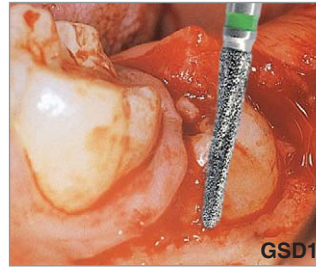
Körnung:

grün/grob = 151 µm – ISO 534

Diamond coated bone cutter for bone incisions, removal and precision finishing works as well as for severing tooth-roots.

Advantages:

The combination of special shaping and diamond coating results in close contact with bone structure, less heat development as well as a better removal of bone chips and tooth structure.

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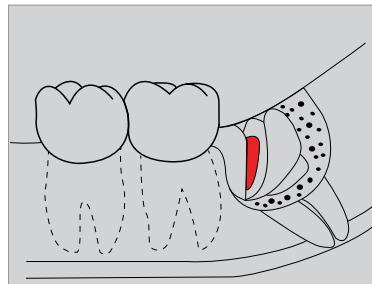
Surgery diamond instrument used for removal and smoothing of bone surfaces as well as separation of teeth and effective machining of the tooth structure in tooth osteotomy.

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
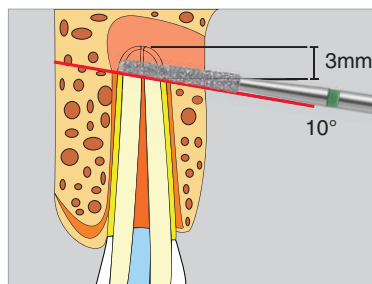
Surgery diamond instrument used for atraumatic separation of teeth and effective machining of the tooth structure during apicectomy. Not recommended for machining bone.

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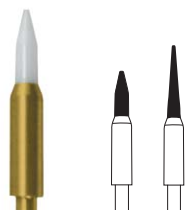300.000 - 500.000

Tissue Trimmer STT:

For gingival treatment. As an alternative to using scalpel or electro-surgery.

Designed for soft-tissue surgery, the STT ceramic tissue trimmer is a rotary instrument that can be used as an alternative to electrosurgery and surgical blades. The biocompatible hard oxide point is said to cut tissue without burning.

Applications include modeling gingiva, opening the sulcus for impression, freeing deep cavities, lacerating inter-radicular granulation-tissue, removing gingival hyperplasia/papillectomies, and exposing intra-osseous implant sections and retinated teeth.

Advantages:

The special porcelain of the working section separates the tissue and reduces bleeding (coagulation).

Reduced heat build-up prevents necroses during treatment.

The high stability of the Soft Tissue Trimmer allows rapid, safe gingival preparation.

To be used without any water/spray cooling!

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